


ChloroPrep[®]

(2% chlorhexidine gluconate & 70% isopropyl alcohol)

Just PINCH-PRESS-PREP!

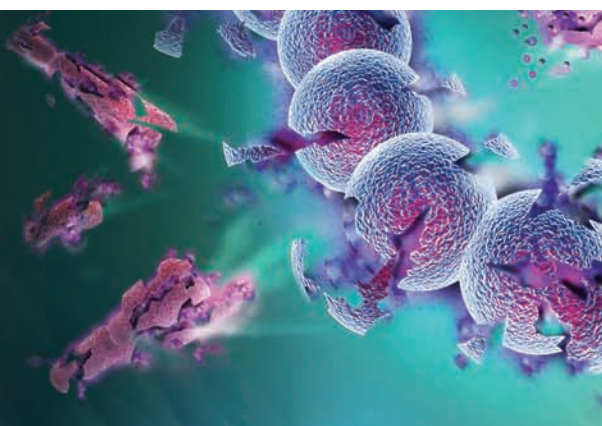

26 mL Hi-Lite Orange™ Applicator


Approximate coverage area: 13.2" x 13.2"
 Catalog number 260815 (Hi-Lite Orange™) (25 applicators per case)

Instructions for use:



PINCH

- Hold applicator as shown; do not touch sponge
- Pinch lever once until you hear a pop to break ampules

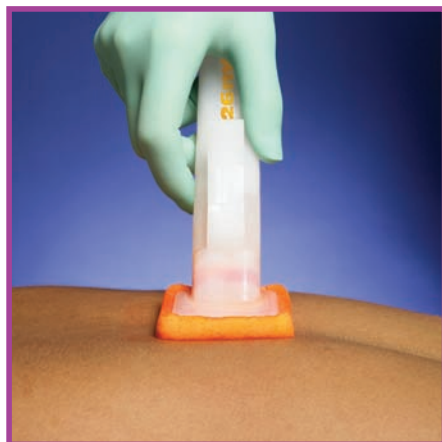

PRESS

- Allow solution to partially load in sponge and gently press applicator against treatment area to evenly distribute solution throughout sponge



PREP

- Once solution is visible on the skin, use a back-and-forth motion to prep the incision site for at least 30 seconds, then continue to work outward to the peripheral area:
- For dry surgical sites (eg, abdomen or arm): Use repeated back-and-forth strokes of the sponge for approximately 30 seconds
 - For moist surgical sites (eg, inguinal fold or axilla): Use repeated back-and-forth strokes of the sponge for approximately 2 minutes
 - Dry time: Allow the area to air-dry for approximately 3 minutes on dry and moist sites
 - Do not blot or wipe away
 - Discard the applicator after a single use

The Hi-Lite Orange™ Tint:

The Hi-Lite Orange Tint in ChloroPrep is Food, Drug, and Cosmetic (FD&C) Yellow #6 dye and is "Generally Recognized As Safe" (GRAS) by the Food and Drug Administration (FDA). The Hi-Lite Orange Tint allows for easy visualization on a variety of skin tones.

How to Remove ChloroPrep[®] with Hi-Lite Orange™ Tint:

It is recommended that ChloroPrep remain on the skin, especially at the incision site, post-procedure, to provide continued antimicrobial activity. The tint will slowly fade from the skin. If policy and procedure for your institution recommends antiseptic removal as part of surgical site cleansing after stitching, we recommend that a clear ChloroPrep applicator be used to maintain optimal persistence. Removal can also be achieved with soap and water or alcohol.

Safety points:

- Do not drape until skin preparation is completely dry
- Do not allow the solution to pool
- Remove any soaked materials, drapes, and gown before using an ignition source
- When prepping the neck, place towels under each side to absorb excess solution. Remove these towels before draping
- Allow more time for solution to dry if solution gets into hair

Additional information:

- For external use only
- Keep solution out of eyes, ears, and mouth
- Do not use on children less than 2 months of age
- Store from 20°C to 25°C (68°F–77°F). Avoid freezing and excessive heat above 40°C (104°F)
 — Storing ChloroPrep outside the recommended temperature range may cause an inconsistent appearance in the Hi-Lite Orange Tint

ChloroPrep representative:

In-service dates:

REORDER #:

Location of ChloroPrep applicators at this facility:

Before using the ChloroPrep 26 mL Hi-Lite Orange™ Applicator, please read the instructions on the package. Use in accordance with the policies and procedures of your hospital.

For more information, visit chloroprep.com or call 800-523-0502.

ChloroPrep is a registered trademark of Medi-Flex, Inc.
 © 2006 Medi-Flex, Inc. MRK-06566

ChloroPrep[®]
 Patient Preoperative Skin Preparation
 2% CHG & 70% IPA

